English	Instructions for use: Dental Instruments	3
Español	Instrucciones de uso: Instrumentos Dentales	5
русский	Инструкция по применению: Зубоврачебные инструменты	8
Deutsch	Gebrauchsanweisung: Zahnmedizinische Instrumente	11
Français	Instructions d'utilisation: Instruments Dentaires	14
Italiano	Istruzioni per l'uso: Strumenti Dentali	17
Português	Instruções de utilização: Instrumentos Dentários	20
Türkçe	Kullanım talimatları: Dental Aletler	23
简体中文	使用说明: 牙科器械	26
日本語	使用説明書: 歯科用機器	28
한국어	적응증: 치과 기구	31
عربي	إرشادات الاستخدام: أدوات طب الأسنان	34
Polski	Instrukcja obsługi: Narzędzia Dentystyczne	36

manufactured by:  
BioHorizons

2300 Riverchase Center  
Birmingham AL, 35244 USA  
TOLL FREE 888.246.8338  
TEL 205.967.7880 FAX 205.870.0304  
www.biohorizons.com

EU Authorised Representative:  
QUALITY FIRST INTERNATIONAL  
Suites 317/318 Burford Business Centre  
11 Burford Road, Stratford  
London E15 2ST United Kingdom  
Telephone +44-208-221-2361  
Telefax +44-208-221-1912

The symbol table below is for reference only. Refer to product packaging label for applicable symbols.

Symbol	Symbol Description
	Caution
	Electronic instructions for use
	Manufacturer
	BioHorizons products carry the CE mark and fulfill the requirements of the Medical Devices Directive 93/42/EEC
	Reference/ article number
	Lot/ batch number
	Do not re-use
	Use-by-date
	Sterile by gamma irradiation
	Date of manufacture
	Caution: U.S. Federal law restricts these devices for sale, distribution and use by, or on the order of, a dentist or physician
	Home position

## ENGLISH

This document supersedes all prior revisions. Original language is English.

### **DESCRIPTION**

BioHorizons instruments are used for dental implant procedures such as site development, implant placement, and implant restorations within the specific indications of each implant system. The label on each instrument contains important product information including whether the instrument is supplied sterile or non-sterile and if the instrument is for single-use only. Instruments and kits supplied non-sterile must be cleaned and sterilized prior to first use and each use thereafter.

### **INDICATIONS FOR USE**

BioHorizons instruments and surgical kits are indicated for use in site development, placement, and restoration of BioHorizons implants and associated components. The BioHorizons Single Use Drills are indicated for use with the BioHorizons Tapered Internal and Laser-Lok 3.0 implants only. No attempt should be made to re-use or re-sterilize the drills. BioHorizons assumes no responsibility for attempted re-use or re-sterilization between patients.

### **CONTRAINDICATIONS**

BioHorizons instruments should not be used with patients that have allergies to the specific materials used, including stainless steel and titanium alloy.

### **DIRECTIONS FOR USE**

Proper surgical procedures and restorative techniques are the responsibility of the medical professional. Each clinician must evaluate the appropriateness of the procedure used based on personal medical training and experience as applied to the patient case at hand. BioHorizons strongly recommends completion of dental implant courses and strict adherence to the instructions pertaining to BioHorizons products.

Internal Abutment-level Driver, Hex-chuck Handpiece: Use only with compatible W&H Hexagon Chucking System handpieces. Drills and Taps: Should be replaced when wear is noticed, such as a decrease in cutting efficiency or when signs of discoloration appear. Cutting instruments should be replaced after approximately 12 to 20 osteotomy cycles, depending on bone density. BioHorizons recommends use of a drill usage chart to track drill use and to ensure drills are replaced as directed. Single Use Drills: Insert the drill into a handpiece and twist protective tubing off prior to use. CGS Drills and Guides: Care must be taken to ensure the CGS Drills follow the angle of the guide handle inserts or sleeves to prevent drill binding. Refer to the drill report for proper drill selection and usage. Site Dilators: Push the dilator into the bone while rotating it. In firmer bone, it may be necessary to use a mallet to tap the dilator to proper depth. Laser-Lok Protective Sleeves: Secure Titanium Base Abutments onto the sleeve side labeled Hybrid or Custom Titanium Abutments onto the side labeled Full Ti to protect Laser-Lok region during bonding and polishing.

The BioHorizons Simple Solutions Replica Lab Tools are hand instruments to aid in the fabrication of burnout wax copings for Simple Solution restorative protocols only. They may be used in the dental suite or lab and mimic the geometry and size of the corresponding Simple Solution Abutment and/or Replica without retentive snap feature, allowing the wax coping(s) to be removed without deformation after hardening. The replica lab tools are manufactured of titanium alloy.

### **WARNINGS AND PRECAUTIONS**

Clinician judgment, as related to individual patient presentations, must always supersede recommendations in any BioHorizons Instructions for Use (IFU). Refer to OSHA standard 29CFR1910.1030 prior to cleaning and sterilization. Additional technical

information is available upon request from BioHorizons, or may be viewed and/or downloaded at [www.biohorizons.com](http://www.biohorizons.com). Contact BioHorizons Customer Care or your local representative with any questions you have regarding a specific IFU.

Prosthetic Instruments - Only use a torque wrench for final installation of prosthetic components requiring a specified installation torque. Finger-tightening of prosthetic components can result in insufficient torque and eventual loosening of the component. Over-tightening prosthetic components could break the component or spin the mating implant.

### **COMPLICATIONS AND ADVERSE EFFECTS**

The risks and complications with instruments, prosthetics, and implants include, but are not limited to: (1) allergic reaction(s) to implant and/or abutment material; (2) implant and/or abutment breakage; (3) abutment screw and/or retaining screw loosening; (4) infection requiring revision of the dental implant; (5) nerve damage that could cause permanent weakness, numbness, or pain; (6) histologic responses possibly involving macrophages and/or fibroblasts; (7) formation of fat emboli; (8) implant loosening requiring revision surgery; (9) maxillary sinus perforation; (10) labial or lingual plate perforation; and (11) bone loss possibly resulting in revision or removal.

### **HANDLING AND STERILIZATION**

Always handle the product with powder-free gloves and avoid contact with hard objects that may damage the surface. If the product is supplied sterile, it should be considered sterile unless the package has been opened or damaged. Using accepted sterile technique, remove product from the package only after the correct size has been determined. If the product is supplied non-sterile or when reprocessing: Remove and discard any shipping material before initial sterilization. Assembled instruments (e.g. OD Secure Cap Tool ODSCT) should be disassembled before each cleaning and sterilization cycle to avoid debris encapsulation, material discoloration and/or inappropriate drying of components. The device must be thoroughly cleaned. Remove any visible debris from the instruments and surgical kit using a soft bristle brush and a broad spectrum cleaning or disinfecting agent such as Hu-Friedy's Enzymax® or equivalent. Rinse thoroughly. Place the instruments in a beaker of the same solution and sonicate for 10 minutes. Rinse thoroughly. Rinse instruments with isopropyl alcohol to remove any soap residue and minerals. Blot instruments with a lint free towel and air dry completely. (For AS123 Hand Unit: blow pressurized air through one of the four drain holes to purge. CGS Quick Connect Handle: blow pressurized air through one end to purge. For both instruments: use a medical instrument lubricant such as STERIS' Hinge Free® after each use.) Refer to the labeling of the cleaning/lubricating agent used for instructions for use. Instruments that are unable to be cleaned, discolored, do not properly interface with mating components, and/or do not articulate as designed should be disposed of. Disassemble the surgical kit and wash the empty tray using the same solution. Rinse with water and dry thoroughly. Return the instruments to the appropriate locations in the tray. Place product in a FDA cleared sterilization bag or wrap and run through one of the following qualified sterilization cycles: 1. Prevacuum Steam: 132°C (270°F) for five (5) minutes minimum. 2. Gravity Steam: 132°C (270°F) for thirty (30) minutes minimum. 3. Gravity Steam: 121°C (250°F) for sixty (60) minutes minimum. 4. Prevacuum Steam: 134°C (273°F) for three (3) minutes minimum. Dry 20 to 50 minutes. Do not exceed 134°C (273°F) during sterilization or drying.

**Attention!** Improper cleaning may lead to inadequate sterilization. Failure to completely dry instruments during autoclaving may leave moisture and cause discoloration and oxidation. The use of hydrogen peroxide or other oxidizing agents will damage the surface of the instruments. Periodic testing, cleaning, and calibration of the autoclave equipment is recommended to ensure the unit remains in proper working order.

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