

Internal & Single-stage Drill Usage Chart

Description	Part Number	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20
2.0mm Starter Drill	122-103																				
#6 Round Bur	122-106																				
2.5mm Depth Drill	122-225																				
2.5 x 7mm Depth Drill	122-12507																				
2.5 x 9mm Depth Drill	122-12509																				
2.5 x 10.5mm Depth Drill	122-125105																				
2.5 x 12mm Depth Drill	122-12512																				
2.5 x 15mm Depth Drill	122-12515																				
2.0mm Starter Drill, Extended Shank	122-403																				
2.5 x 7mm Depth Drill, Extended Shank	122-42507																				
2.5 x 9mm Depth Drill, Extended Shank	122-42509																				
2.5 x 10.5mm Depth Drill, Extended Shank	122-425105																				
2.5 x 12mm Depth Drill, Extended Shank	122-42512																				
2.5 x 15mm Depth Drill, Extended Shank	122-42515																				
2.5mm Depth Drill, Extended Shank	122-425																				
3.0mm Width Increasing Drill	122-230																				
3.0mm Width Increasing Drill, Extended Shank	122-430																				
Internal 3.5mm Crestal Bone Drill	PCBD35																				
Single-stage 3.5mm Counter-sink, 3.5mm platform	SYCD35																				
Internal / Single-stage 3.5mm Bone Tap	PT35																				
3.4mm Width Increasing Drill	122-232																				
3.4mm Width Increasing Drill, Extended Shank	122-432																				
Internal 4.0mm Crestal Bone Drill	PCBD40																				
Single-stage 4.0mm Counter-sink, 3.5mm platform	SYCD40																				
Single-stage 4.0mm Counter-sink, 4.5mm platform	SGCD40																				
Internal / Single-stage 4.0mm Bone Tap	PT40																				
3.9mm Width Increasing Drill	122-237																				
3.9mm Width Increasing Drill, Extended Shank	122-437																				
4.4mm Width Increasing Drill	122-242																				
4.4mm Width Increasing Drill, Extended Shank	122-442																				
Internal 5.0mm Crestal Bone Drill	PCBD50																				
Single-stage 5.0mm Counter-sink, 4.5mm platform	SGCD50																				
Single-stage 5.0mm Counter-sink, 5.7mm platform	SBCD50																				
Internal / Single-stage 5.0mm Bone Tap	PT50																				
4.9mm Width Increasing Drill	122-247																				
4.9mm Width Increasing Drill, Extended Shank	122-447																				
5.4mm Width Increasing Drill	122-252																				
5.4mm Width Increasing Drill, Extended Shank	122-452																				
Internal 6.0mm Crestal Bone Drill	PCBD60																				
Single-stage 6.0mm Counter-sink, 5.7mm platform	SBCD60																				
Internal / Single-stage 6.0mm Bone Tap	PT60																				

Important Note:
All drills should be replaced after they have been used 12 to 20 times, depending upon bone density.

Shop on-line at www.biohorizons.com